


# VitaScan Calibration Test Tool Setup

1

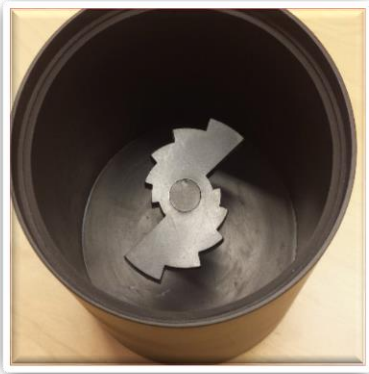

Please follow this VitaScan Calibration Tank Setup before performing a calibration.

2



Rinse the tank thoroughly before adding water. After rinsing inspect the tank and ensure it is clean.

3



Align the target with the target template.

See picture.

NOTE: Target might get unaligned during transportation.

# VitaScan Calibration Test Tool Setup

4



Gently pour sterile or pre-boiled water in to the tank.  
Fill until the inner edge.

After water is added, wait at least 5 minutes before proceeding.

NOTE: It is important to wait, this eliminates air bobbles in the water.

CAUTION: Do not use hot water.

5



Place the lid firmly on the tank, with the probe holder up.

6



Insert the probe into the lid, ensure the probe leans against the probe holder.

See picture.